

## AURORE PHOTOMASK


### ANTI-AGE AND LIGHTENING PHOTOACTIVE MASK

#### 1-PRINCIPLE ACTION

This photomask contains active molecules encapsulated in nanosomes, for a deep cutaneous action :

- ATP (Adenosine Triphosphate), in order to provide the necessary energy for cellular chemical reactions,
- Riboses, essential components in metabolic processes,
- Ergothioneine, an intracellular antioxidant.

The MEDISOL® LED emits pulsed photons of red (630 nm) and infrared (850 nm) wavelengths. These will act on the mitochondria to boost cellular respiration. This phenomenon, called photobiomodulation, increases the production of ATP molecules that will stimulate neocollagenesis. The active molecules of the AURORE photomask have a synergistic action with the LED MEDISOL®, to offer an anti-aging effect and boost to the skin.

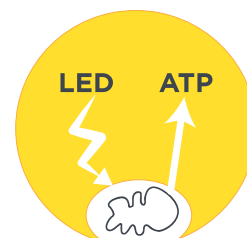

The indications are :

- Rejuvenation
- Wrinkles
- Inflated pores
- Skin tightening.

#### 2-PRE-TREATMENT INSTRUCTIONS

Explain in detail how the AURORE photomask works.

Present the treatment plan to the patient and the results according to the indication, as well as the possible side effects related to the MEDISOL® treatment.

Take before / after treatment pictures according to standardized procedures.

#### 3-APPLICATION OF THE AURORE PHOTOMASK

- Clean the patient's skin.
- Open the package with scissors, taking care not to damage the mask.
- Remove the protective film from both sides of the mask.
- Place the most impregnated side of the mask on the face and adjust it.
- After placing the protective shells on the patient's eyes, use the MEDISOL® LED lamp and follow the predefined protocol AURORE Photomask.
- At the end of the treatment, remove the mask and gently massage the face.



300, rue Isaac Newton  
Technoparc Epsilon 1  
83700 SAINT RAPHAËL  
FRANCE

04 94 49 49 45  
contact@deleo-shop.com  
[www.deleo-shop.com](http://www.deleo-shop.com)

## 4-PROTOCOL

The protocol to use in combination with the Aurore photomask is «AURORE Photomask». This protocol is pre-registered in the MEDISOL® LED lamp, just launch it.

TIME RANGE 1				TIME RANGE 2				TIME RANGE 3			
Colors	Energy	Time	Pulse	Colors	Energy	Time	Pulse	Colors	Energy	Time	Pulse
Yellow	1 J	4 min	Long	Red IR Yellow	24 J 4 J 1 J	12 min	Continu	Red IR	2 J 6 J	4 min	Long

## 5-POST-TREATMENT INSTRUCTIONS

- The patient can resume normal activities following the session.
- Apply a moisturizer morning and evening for 2 to 3 days. When fully absorbed, apply a minimum SPF 30 sunscreen, if sun exposure.
- For 48 hours after the treatment, avoid the use of any product containing essential oils, acids or retinoids.

## 6-EXPECTED RESULTS

The skin is smoothed and recovers its radiance. It gains firmness and elasticity.

Wrinkles are reduced by following a complete treatment plan, which consists of 6 to 8 sessions spaced 3 to 7 days apart.

Like any cosmetic medicine treatment, the clinical results will vary from one person to another and no guarantee can be given regarding the results.

## 7-PRECAUTIONS AND WARNINGS

- Avoid contact with eyes.
- Discontinue use immediately if redness or itchy irritations appear.
- CONTRAINDICATIONS: Refer to the general contraindications of the MEDISOL® LED lamp.

If you have any questions about the AURORE photomask, please contact DELEO SHOP at the email address [contact@deleo-shop.com](mailto:contact@deleo-shop.com) or by phone at 04 94 45 83 75.

Distributed in Australia by:



For more information please contact:  
Gavin Dimitri: 0451 308 380 or [gavin@i-medix.com.au](mailto:gavin@i-medix.com.au)  
[www.i-medix.com.au](http://www.i-medix.com.au)

300, rue Isaac Newton  
Technoparc Epsilon 1  
83700 SAINT RAPHAËL  
FRANCE

04 94 49 49 45  
[contact@deleo-shop.com](mailto:contact@deleo-shop.com)  
[www.deleo-shop.com](http://www.deleo-shop.com)